



Designed with surgeons and surgical technicians
to meet their need for **safety without compromise.**

Americas

3883 Rogers Bridge Road NW
Suite 501
Duluth, GA 30097
Tel: +1 770 448 9493

Asia


15 Hoe Chiang Road #12-02
Tower Fifteen
Singapore 089316
Tel: +65 63451588

Europe


Tel: +44 330 122 2630

A product of **MediPurpose™**
www.anzenscalpel.com

1 Attaching Blade Cartridge

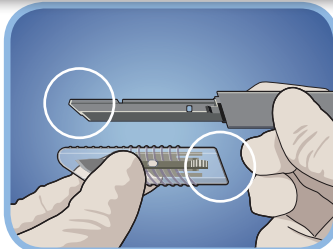


Align Handle and Cartridge orientations by matching end angles.

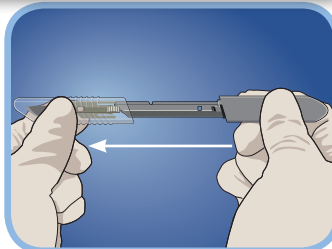


Insert Handle into Cartridge.

2 Extending Blade

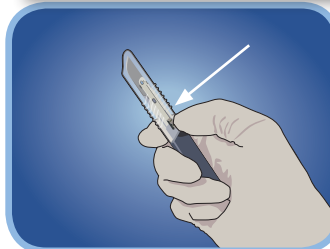


Depress Button on Cartridge.

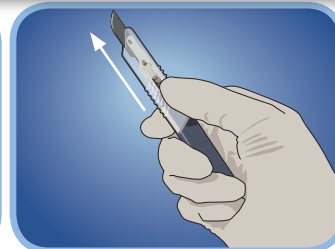


Slide Button forward until it locks.

3 Retracting Blade

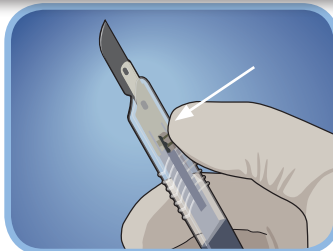


Depress Button on Cartridge.

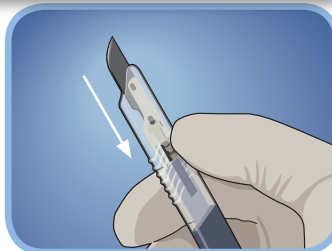


Slide Button back until it locks.

4 Detaching Blade Cartridge

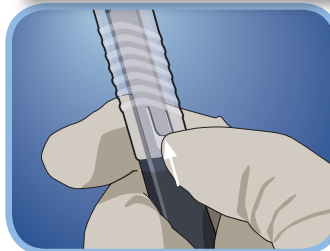


Lift Release Tab (dorsal view).

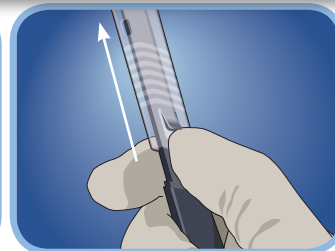


Slide Cartridge forward to eject.

Ventral View (Button Side)

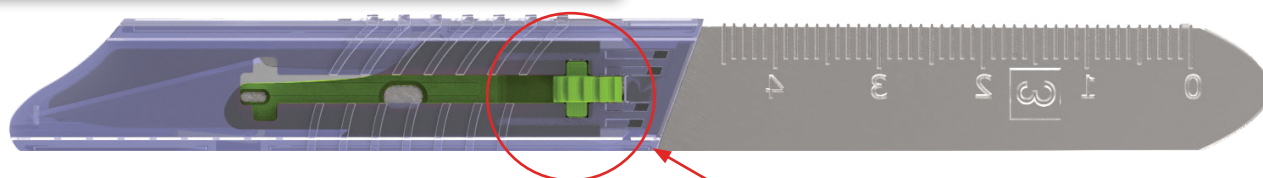

Dorsal View (Release Tab Side)

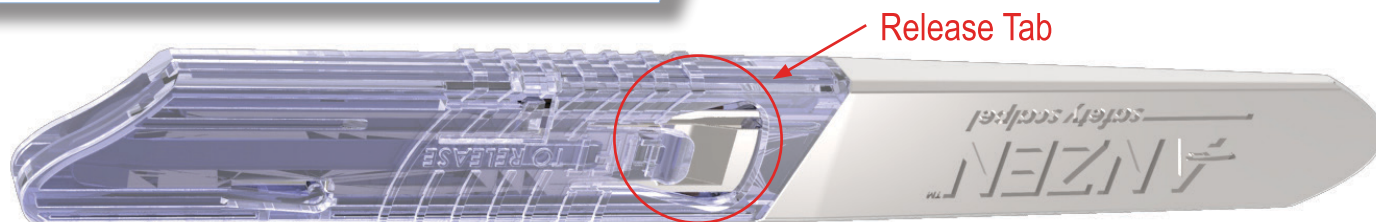

Americas

3883 Rogers Bridge Road NW
Suite 501
Duluth, GA 30097
Tel: +1 770 448 9493

Asia

15 Hoe Chiang Road #12-02
Tower Fifteen
Singapore 089316
Tel: +65 63451588

Europe

Tel: +44 330 122 2630

A product of **MediPurpose™**
www.anzenscalpel.com