





**1** Clean test site with isopropyl alcohol and air dry. Remove protective cap from safety lancet.

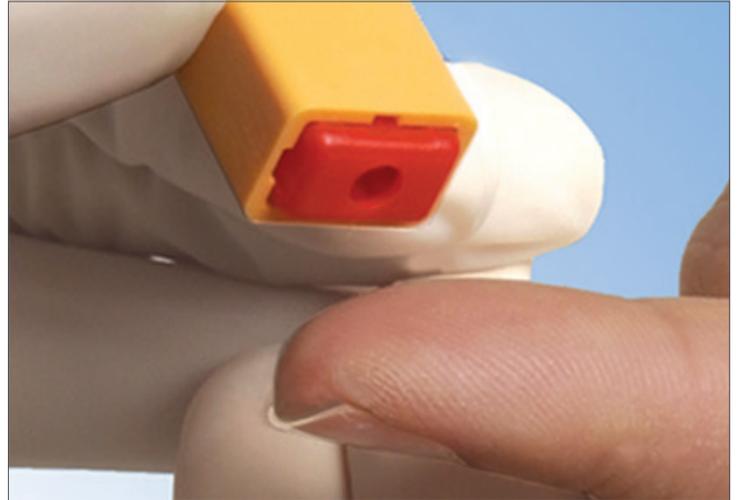

**2** Place red end of safety lancet on the test site.



**3** Push the safety lancet down against test site to activate lancet mechanism.

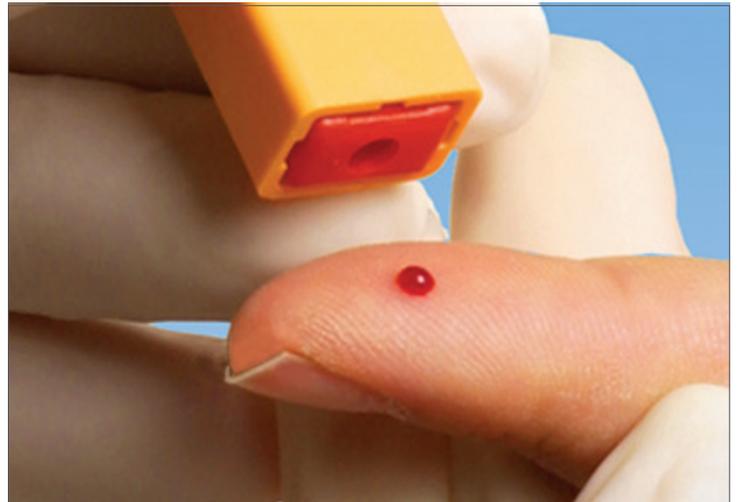

**4** Once used, simply discard lancet into a sharps container.

1. Ensure hands are washed properly before and after use of the device.
2. For safety and to maintain sterility, do not push the red raised platform onto patient's finger until ready to obtain a blood sample.
3. The Surgilance™ safety lancet is designed for healthcare professionals to be used in health care settings. At-home users who are physically and mentally able to use the device for self-testing reasons should consult their healthcare provider before use.
4. If used in health care settings, ensure lancets are used with appropriate personal protective equipment e.g. gloves.
5. Do not use if the package is broken.

6. Do not store in high temperature, high humidity environment or with erosive gas.
7. Keep dry and good aeration.
8. Do not use if product is damp or expired.
9. Warnings: The device is not intended for reuse. Do not use each device on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested.
10. In the event of a serious incident, contact MediPurpose at [sales@medipurpose.com](mailto:sales@medipurpose.com) and/or the competent authority of the Member State in which the incident occurred.