



THIS  
IS A  
**babyLance™**  
BABY

babyLance™ Safety Heelstick User Guide

**Designed for neonatal nurses...with neonatal nurses!**

babyLance™ safety heelsticks have earned worldwide popularity for their **smooth and accurate incisions**, **ease of use** and **safety**.





1  
Select an incision site on the flat bottom surface of the heel, then clean the area.



2  
Remove the Trigger Lock, but do not pull back the Trigger until ready for use.



3  
Align the Blade Slot's visual marking with the incision site and pull the Trigger back with your index finger. Discard.



4  
Gently wipe away the first droplet of blood, then collect the desired quantity. That's it!



**Americas**  
3883 Rogers Bridge Road NW  
Suite 501  
Duluth, GA 30097  
Tel: +1 770 448 9493

**Asia**  
15 Hoe Chiang Road #12-02  
Tower Fifteen  
Singapore 089316  
Tel: +65 63451588

**Europe**  
Tel: +44 330 122 2630

**MediPurpose**<sup>™</sup>  
medipurpose.com/babylance